# MARSDEN

## **USER MANUAL**

**BAS-250** 

Please take time to read these instructions before starting to use the scale

## Contents

Introduction	3
Product Specification	3
Safety Instructions	2
Operation	5

Thank you for purchasing a Marsden professional baby scale. This is a precision weighing instrument and considerate use will result in many years of accurate weighing.

The scale has a maximum load capacity of 25kg which must not be exceeded.

Marsden's policy is one of continuous development and therefore we reserve the right to change product specification without notice.

## **Product Specification**

Model	BAS-250
Accuracy Class	N/A
Capacity/Division	25kg x 10g
Weight of Scale	Approximately 2kg
Units of Measure	Kg/lb
Function Keys	ON/OFF/TARE, HOLD, UNIT
Stabilization Time	1-2 Seconds
Operating Temperature	0 to 40°C
Power Supply	2 x AA batteries
Dimensions	630mm x 400mm

For Class III Approved baby scales, visit www.marsden-weighing.co.uk/medical-scales/baby-scales.html.

## Safety Instructions

Before putting the device into use, please read with care the information given in this user manual, which contains important instructions for proper installation, use and maintenance of the device.

Marsden/the manufacturer shall not be liable for damages arising from failure to heed the following instructions:

- When using electrical components under increased safety requirements, always comply with appropriate regulations.
- Inappropriate installation/use will render the warranty null and void.
- Ensure the voltage marked on the power supply unit matches your mains supply.
- This device is designed for use indoors.
- Observe the permissible ambient temperatures for use.
- The device meets the requirements for electromagnetic capability. Do not exceed the maximum values specified in the applicable standards.
- Batteries should be kept away from small children. If swallowed, promptly seek medical assistance.

If you have any problems, contact Marsden/your local dealer/your service partner.

## Cleaning

- We recommend using alcohol-based wipes or similar when cleaning the scale.
- Please do not use large amounts of water when cleaning the scale as this will cause damage to the inner electrics. Please refrain from using corrosive liquids or high pressure washers.

#### Maintenance

- The scale does not require any routine maintenance. However, we recommend checking the scale's accuracy at regular intervals. For information regarding Marsden service contracts call 01709 364296.
- If any inaccuracies occur, please contact Marsden/your local dealer/your service partner.

## Disposing of the Scale

- This product should not be treated as regular household waste, but should be handed in to an
  electrical/electronic equipment recycling centre.
- You can obtain further details from your local council or your municipal waste disposal company.

Please note: Images in this manual are for illustration purposes only.


## Operation

#### Switching on the Scale



Press the ON/OFF/TARE button and the display will show a zero reading.

#### Switching off the Scale

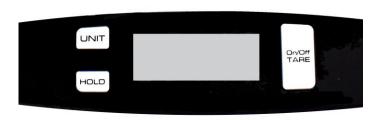

Press the ON/OFF/TARE button when the scale is turned on. The scale will now power down.

#### Setting the Scale to Zero



If for any reason the scale shows a reading other than zero with nothing on the scale it can be reset.

Press the ZERO key once.

#### **Hold Function**



To get a stable weight reading when a baby is on the scale, press the HOLD button once.



After a few seconds the scale will lock on the baby's weight. The weight will hold on the display, even after you take the baby off the scale.



Press ON/OFF/TARE to return the scale to zero.

#### Switching between KG and lb



To switch weight readings between KG and lb, press the UNIT button. The units will show on the right hand side of the display.

#### **Using Tare**



If you need to remove the weight of an item from the display, for example a baby's blanket, add the item to the scale.



Press ON/OFF/TARE once.



Add the baby and the blanket to the scale for an accurate reading of the baby only.



Tel: 01709 364296 / 0800 169 2775

Fax: 01709 364293

E-mail: sales@marsdengroup.co.uk

### Manufacturing and Distribution:

Unit 7, Centurion Business Park,

Coggin Mill Way,

Rotherham,

S60 1FB

#### **Head Office:**

Unit 1, Genesis Business Park,

Sheffield Road,

Rotherham

S60 1DX

www.marsden-weighing.co.uk