# MARSDEN



## Contents

Product Specification	3
Safety Instructions	4
Before Using the Scale	5
Display & Function Keys	5
Operation	6
Error Messages	7



#### **Manufacturer's Declaration of Conformity**

This product has been manufactured in accordance with the harmonized European standards, following the provisions of the below stated directives:

Electro Magnetic Compatibility Directive 2014/30/EU

Low Voltage Directive 2014/35/EU

Marsden Weighing Machine Group Ltd.

## **Product Specification**

Thank you for purchasing a Marsden professional veterinary scale.

This is a precision weighing instrument and considerate use will
result in many years of accurate weighing.

The scale has a maximum load capacity of 20kg which must not be exceeded.

Model	Marsden V-20	
Capacity/Division	20kg x 10g	
Units of Measure	Kg/lb/lb:oz	
Function Keys	MEMORY, KG/LB, TARE/ON/OFF	
Display	Easy to read 2cm LCD display with four active digits	
Operating Temperature	5 to 35°C	
Power Supply	9v battery 9v 100mA DC adaptor	
Dimensions	With handles: 620mm x 350mm x 50mm Platform only: 500mm x 350mm x 50mm	

Please note: Marsden's policy is one of continuous development and therefore we reserve the right to change product specification without notice.

Images in this manual are for illustration purposes only.

#### Safety Instructions

Before putting this scale into use, please read with care the information given in this user manual, which contains important instructions for proper installation, use and maintenance of the device.

Marsden/the manufacturer shall not be liable for damages arising from failure to heed the following instructions:

- When using electrical components under increased safety requirements, always comply with appropriate regulations.
- Inappropriate installation/use will render the warranty null and void.
- Ensure the voltage marked on the power supply unit matches your mains supply.
- This product is not waterproof. We do not recommend using the scale in humid, damp or wet environments.
- Observe the permissible ambient temperatures for use.
- The device meets the requirements for electromagnetic capability. Do not exceed the maximum values specified in the applicable standards.

If you have any problems, contact Marsden/your local dealer/your service partner.

## Cleaning

- Please do not use corrosive liquids or large amounts of water to clean the scale. We recommend using alcohol-based wipes or similar.
- Always disconnect the scale from the mains power supply before cleaning.

⚠ Never disassemble the scale as this may cause malfunction.

#### Maintenance

- The scale does not require any routine maintenance. However, we recommend checking the scale's accuracy at regular intervals. If any inaccuracies occur, please contact your local dealer or service partner.
- You can book an accuracy check with the Marsden Service team on 01709 364296.


## Disposing of the Scale

• This product should not be treated as regular household waste, but should be handed in to an electrical/electronic equipment recycling centre.

## Before using the Scale

- All Marsden V-20 scales come complete with plastic platform, plastic base and two soft handles. There are three easy to use function keys: MEMORY, KG/LB, TARE/ON/OFF.
- The scale can be used in a fixed location or as a portable scale, powered by either the DC adaptor or with a 9V battery.
- If you require any assistance with using the V-20, call the Marsden team on 01709 364296, quoting model number V-20 and the serial number, which can typically be found on the underside of the scale.
- Before use, ensure the scale is on a flat surface. An un-level surface for weighing could result in inaccurate readings.

## Display & Function Keys

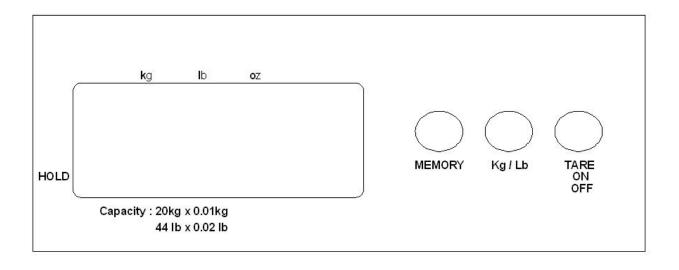

#### Operation

#### Switching the Scale on/off

- To switch the V-20 on, press **TARE/ON/OFF**.
- Once the scale shows a line of zeros, it is ready to use.
- To switch the V-20 off, press and hold **TARE/ON/OFF** for three seconds.

#### Using the Hold function

- The Hold function allows you obtain an accurate weight reading, even if the animal is moving on the scale.
- The Hold function is automatic, meaning the display will lock without prompting.
- The arrow at the bottom left of the display will flash briefly and then remain on the screen, indicating that an accurate weight is being held on the screen.
- To disable the Hold function once you have finished weighing, press **TARE/ON/OFF**.

#### Using the Tare function

- The Tare function allows you to remove unwanted weight from the display, such as a towel or blanket.
- With the scale switched on, place the item you wish to tare off onto the platform.
- Press TARE/ON/OFF. The display will show zero, and when you remove the item, a minus reading.
- When you place an animal to the scale the weight reading will show the weight of the album, minus the weight of the item you removed using Tare.
- To cancel the saved tare weight, switch the scale off.

#### Switching between kg, lb and lb:oz

 At any time, with the scale switched on press KG/LB to toggle between kg, lb and lb:oz readings. The unit of measurement will be indicated at the top of the display.

#### **Using Memory Recall**

- Press **MEMORY** to store a weight reading showing on the indicator.
- With the scale switched on and with a reading below 1kg, press **MEMORY** to recall one previously stored weight reading.

## **Error Messages**

Display	Problem	Solution
LOW	Low battery	Replace the 9V battery is connect the DC adaptor
Err	Over Zero	If zero count is higher than the setting range, the display will show this message. The scale will turn off in 30 seconds if the status continues.
Er-ri	Under Zero	If zero count is below the setting range, the display will show this message. The scale will turn off in 30 seconds if the status continues.
Err	Overload	If the total load exceeds 20kg, the display will show this message. The weight must be removed immediately from the scale to prevent damage.



Tel: 01709 364296 / 0800 169 2775

Fax: 01709 364293

E-mail: sales@marsdengroup.co.uk

#### Manufacturing and Distribution:

Unit 7, Centurion Business Park, Coggin Mill Way, Rotherham, S60 1FB

#### **Head Office:**

Unit 1, Genesis Business Park, Sheffield Road, Rotherham S60 1DX

www.marsden-weighing.co.uk

IN-0806 3022G REV 01/2017