

MARSDEN

USER MANUAL

V-25

Please take time to read these instructions before starting to use the scale



Contents

Product Specification	3
Safety Instructions	4
Before Using the Scale	5
Operation	6
Error Messages	7



Manufacturer's Declaration of Conformity

This product has been manufactured in accordance with the harmonized European standards, following the provisions of the below stated directives:

Electro Magnetic Compatibility Directive 2014/30/EU

Low Voltage Directive 2014/35/EU

Marsden Weighing Machine Group Ltd.

Product Specification

Thank you for purchasing a Marsden professional veterinary scale.
This is a precision weighing instrument and considerate use will result in many years of accurate weighing.

The scale has a maximum load capacity of 25kg which must not be exceeded.

Model	Marsden V-25
Capacity/Division	25kg x 10g
Units of Measure	Kg/lb
Function Keys	ON/OFF/TARE, HOLD, UNIT
Display	Easy to read LCD display with four active digits
Operating Temperature	5 to 35°C
Power Supply	2x AAA batteries
Dimensions	630mm x 400mm
Warranty	One year
Weight of Scale	1.8kg

Please note: Marsden's policy is one of continuous development and therefore we reserve the right to change product specification without notice.

Safety Instructions

Before putting this scale into use, please read with care the information given in this user manual, which contains important instructions for proper installation, use and maintenance of the device.

Marsden/the manufacturer shall not be liable for damages arising from failure to heed the following instructions:

- When using electrical components under increased safety requirements, always comply with appropriate regulations.
- Inappropriate installation/use will render the warranty null and void.
- Ensure the voltage marked on the power supply unit matches your mains supply.
- This product is not waterproof. We do not recommend using the scale in humid, damp or wet environments.
- Observe the permissible ambient temperatures for use.
- The device meets the requirements for electromagnetic capability. Do not exceed the maximum values specified in the applicable standards.

If you have any problems, contact Marsden/your local dealer/your service partner.

Cleaning

- Please do not use corrosive liquids or large amounts of water to clean the scale. We recommend using alcohol-based wipes or similar.
- Always disconnect the scale from the mains power supply before cleaning.
- ⚠ Never disassemble the scale as this may cause malfunction.

Maintenance

- The scale does not require any routine maintenance. However, we recommend checking the scale's accuracy at regular intervals. If any inaccuracies occur, please contact your local dealer or service partner.
- You can book an accuracy check with the Marsden Service team on 01709 364296.

Disposing of the Scale

- This product should not be treated as regular household waste, but should be handed in to an electrical/electronic equipment recycling centre.

Before using the Scale

- Marsden V-25 scales consist of a plastic platform. There are three easy to use function keys: ON/OFF/TARE, HOLD, UNIT.
- The scale can be used in a fixed location or as a portable scale, powered by 2x AAA batteries.
- If you require any assistance with using the V-25, call the Marsden team on 01709 364296, quoting model number V-25 and the serial number, which can typically be found on the underside of the scale.
- Before use, ensure the scale is on a flat surface. An un-level surface for weighing could result in inaccurate readings.

Operation

Switching the Scale on/off

- To switch the V-25 on, press **ON/OFF/TARE**.
- The scale will first show its last recorded weight.
- When the scale shows a line of zeros, it is ready to use.
- To switch the V-25 off, press and hold **ON/OFF/TARE** for three seconds.

Using the Tare function

- The Tare function allows you to remove unwanted weight from the display, such as a small cage.
- Place the object on the scale and press **ON/OFF/TARE**. The scale will process the added weight, display the weight recorded in the memory and display 0:0.0.
- Place your pet on the scale, and only the weight of the pet will be displayed.
- Alternatively, start by pressing the **ON/OFF/TARE** button, place the object on the scale and press **ON/OFF/TARE**. Place your pet on the scale and only the pet's weight will be displayed.

Using the Memory function

- With the scale at zero, place the pet on the scale and the weight will be displayed.
- Push the **HOLD** button to hold the weight, this will then be recorded to the scale's memory.
- The scale will then compare the newly recorded weight to the previous recorded weight reading, and display whether the new reading is 'up' or 'down' on the original.

Error Messages

Display	Problem	Solution
L.O.	Low battery	Replace the 2x AAA batteries
<u>Err</u>	Overload	If the total load exceeds 25kg, the display will show this message. The weight must be removed immediately from the scale to prevent damage.



Accuracy Assured

Tel: 01709 364296 / 0800 169 2775

Fax: 01709 364293

E-mail: sales@marsdengroup.co.uk

Manufacturing and Distribution:

Unit 7, Centurion Business Park,

Coggin Mill Way,

Rotherham,

S60 1FB

Head Office:

Unit 1, Genesis Business Park,

Sheffield Road,

Rotherham

S60 1DX

www.marsden-weighing.co.uk