

# MARSDEN

USER MANUAL

## HM-201D

Please take time to read these  
instructions before starting  
to use the scale



# Contents

Introduction	3
Product Specification	3
How to Install	5
Contact Information	6


Thank you for purchasing a Marsden professional medical height measure. This is an accessory to be used with the Marsden DP-3810 indicator.

Marsden's policy is one of continuous development and therefore we reserve the right to change product specification without notice.

### Product Specification

Model	HM-201D
Measuring Range	120cm – 200cm
Graduations	1mm
Stabilization Time	1 - 2 Seconds
Operating Temperature	0 to 40°C
Dimensions (w x d x h)	1000mm x 75mm x 55mm

The HM-201D Digital Height Measure is an optional accessory for the DP-3810 series. When the person being measured places the underside of the cursor on top of their head the height measurement will be automatically displayed on the screen of the DP-3810 indicator.

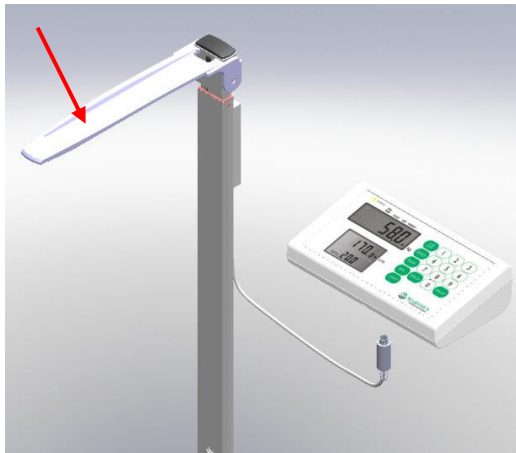


The automatic height measurement also enables the automatic calculation and display of the Body Mass Index (BMI) of the measured person.

## How To Install The HM-201D

The connection point between the DP-3810 indicator and HM-201D is clearly shown in the image below.

Insert the cable from the HM-201D height measure into the socket on the underside of the DP-3810 indicator before use.

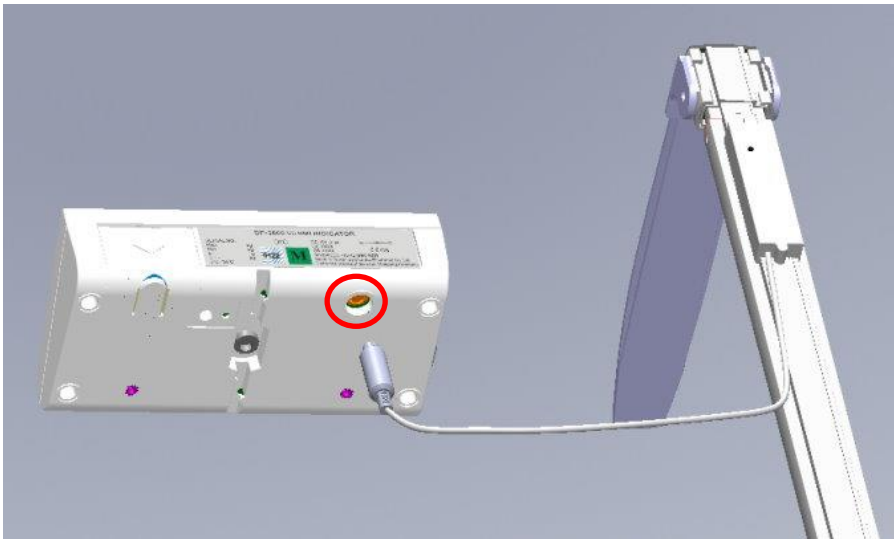

# MARSDEN

Tel: 01709 364296 / 0800 169 2775

Fax: 01709 364293

E-mail: [sales@marsdengroup.co.uk](mailto:sales@marsdengroup.co.uk)

## Manufacturing and Distribution:

Unit 7, Centurion Business Park,

Coggin Mill Way,

Rotherham,

S60 1FB

## Head Office:

Unit 1, Genesis Business Park,

Sheffield Road,

Rotherham

S60 1DX

[www.marsden-weighing.co.uk](http://www.marsden-weighing.co.uk)