

# MARSDEN

## OPERATING MANUAL

### M-150

Please take time to read these instructions before starting to use the scale



# MARSDEN

## Contents

Maintenance and General Care	3
Assembly Instructions	4
Operating Instructions	6
BMI Operation	7
Zero/Tare/Hold/Units	8
Error Messages	9
Contact Details	10

# MARSDEN


## Maintenance and General Care

- Repairs and servicing should only be carried out by an authorised service agent.
- There are no user serviceable parts in the weighing scale itself.
- The mains power adaptor fuses are not accessible.
- When the adaptor is plugged into the mains supply a red LED should illuminate to indicate that the adaptor is functioning correctly. If the LED does not illuminate, then the scale should be checked by competent personnel.
- The weight indicator contains an internal rechargeable battery pack; this should only be replaced by authorised service agents. Under normal operating conditions the batteries should last for several years.
- We would recommend using alcohol based wipes or similar when cleaning the scale.
- Please do not use large amounts of water when cleaning the scale as this could cause damage to the scale's electronics. You should also refrain from using corrosive liquids or high pressure washers.
- Always disconnect the scale from the mains power supply before cleaning.


# MARSDEN

## Assembly Instructions

- 1) Open the packaging and check that all parts are accounted for.

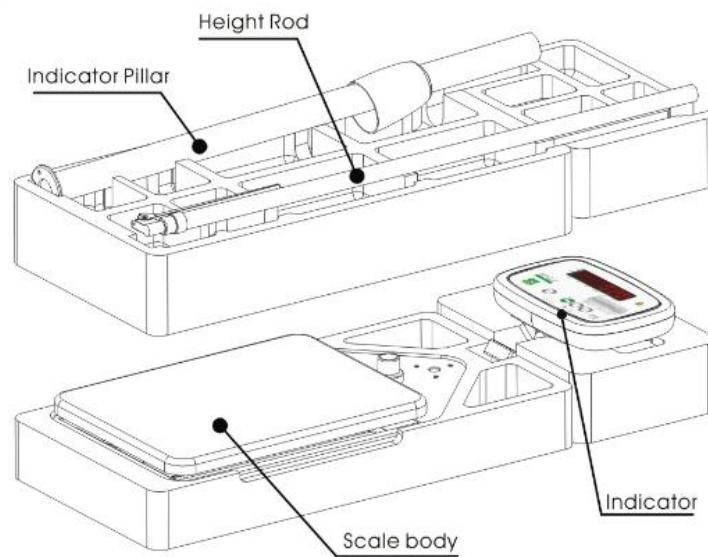


- 2) Place the cable through the column, then fix the indicator to the pillar as shown in the image below.

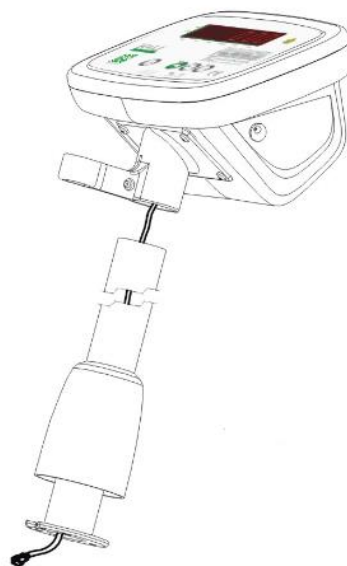


# MARSDEN

- 3) Connect the cable from the indicator into the load cell cable in the base.

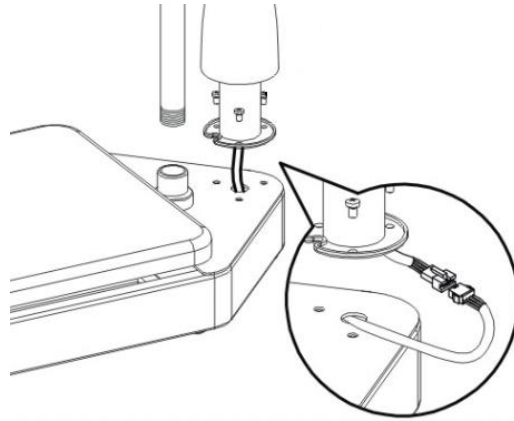

- 4) Fix the column onto the scale base using the 3 x M5 bolts, making sure not to trap any wires.

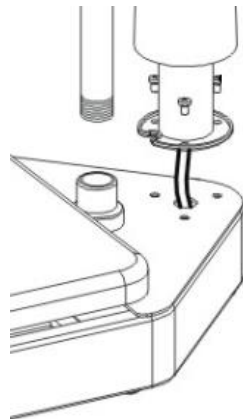


- 5) Push the height rod through the hole at the base of the indicator and then connect to the black stopper on the base of the scale. Rotate the height measure clockwise to screw into the base.



# MARSDEN


## Operating Instructions

### 1) Turning the scale on and off

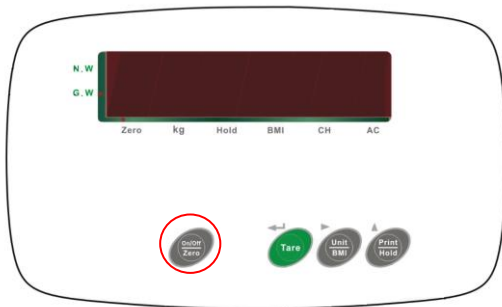

To turn the scale on press the ON button firmly.


The model number and software version will briefly show on the display. Then the scale will display 0.0 and it is ready to use.

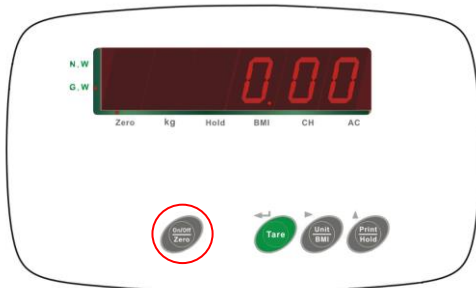


To turn the scale off, press and hold the OFF button for 3 seconds.

# MARSDEN

## 2) BMI Operation

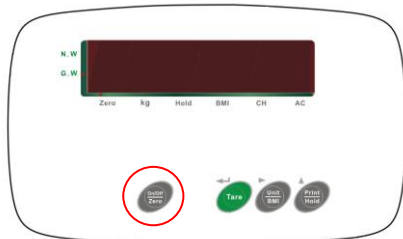


Turn the scale on and then allow the individual to stand on the scale.

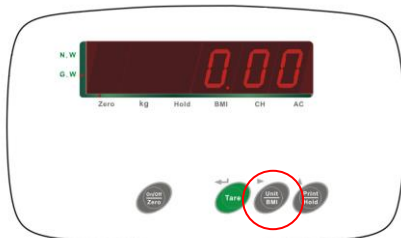

When a stable weight is showing, firmly press the BMI button.

The display will show H 1.75. This is the height in metres and centimetres.

The last digit should be flashing.

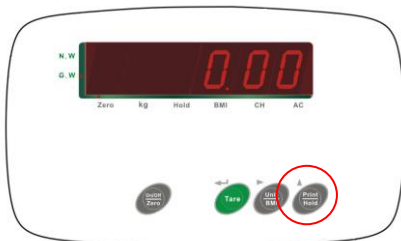


Press the BMI key again to move to the digit that you need to change.

Press the PRINT key to change the flashing digit (from 0-9).



When height has been inputted press the TARE button to confirm. The BMI will then display.

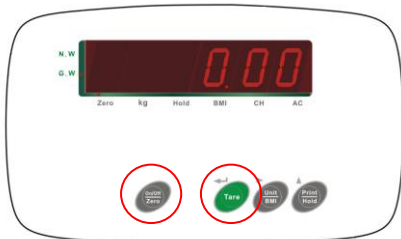

To exit BMI mode, press TARE or ZERO.

Normal weighing mode will then be resumed.

# MARSDEN

## 3) Zero/Tare/Hold/Units




To make the scale show 0.00, press the ZERO button once.



If you want to remove unwanted weight, such as a coat for example, put the coat on the scale and press the TARE button.

When the coat is removed from the scale a minus weight will be displayed. An individual can then stand on the scale and the reading will be accurate.

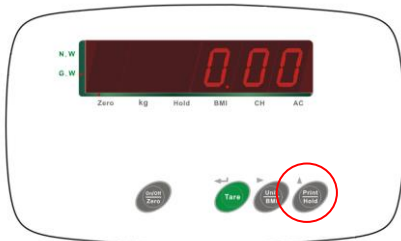

To get a stable weight reading press the HOLD key.

This will capture the weight and display it even if the individual steps off the scale.

To cancel, press the HOLD key again.



To change the unit of measurement in standard weighing mode, press the UNIT key for 3 seconds with nobody on the scale.

The indicator will then change the unit between kg and lb.



# MARSDEN

## Error Messages

Error Message	Meaning
Err1	Incorrect password has been entered
Err2	Out of setting range
Err4	Unable to zero due to the weight being out of zero range

# MARSDEN

<b>Head Office: Sales, Service, Marketing &amp; Accounts</b>	Unit 1, Genesis Business Park, Sheffield Road, Rotherham, S60 1DX
<b>Telephone:</b>	0845 130 7330
<b>Fax:</b>	0845 130 7440
<b>Despatch, Production &amp; Technical Support:</b>	Unit 7, Centurion Business Park, Coggin Mill Way, Rotherham S60 1FB
<b>Telephone:</b>	01709 364296
<b>Fax:</b>	01709 364293